# IT - ELI Spreader SS e Ni-Ti

## Strumenti canalari manuali

### Descrizione

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e-mail: info@dental-world.it / web: www.dental-world.it

Eli-Dent Group S.p.A.

Via Guido Rossa, 24 A/B/C - 50032

Borgo San Lorenzo (FI) - www.elidentgroup.it

Tel. +39 055 600545 - Fax +39 055 601069

# EN - ELI Spreader SS e Ni-Ti

## Manual root canal instruments

### Description

ELI Spreader are steel and nickel titanium instruments with a conical shape and a sharp tip.

Suitable for lateral condensation of gutta-percha.

### Indications

For use in dental practices or hospitals by qualified personnel only.

### Field of application

For lateral condensation of gutta-percha.

### Measures

ELI Spreader - Stainless Steel		
	21mm	25mm
○ 15	7300630S	7300730S
● 20	7300631S	7300731S
● 25	7300632S	7300732S
● 30	7300633S	7300733S
● 35	7300634S	7300734S
● 30	7300635S	7300735S
15-40	7300636S	7300736S

ELI Spreader - Nickel-Titanium	
	25mm
○ 15	7300740S
● 20	7300741S
● 25	7300742S
● 30	7300743S
● 35	7300744S
● 30	7300745S
15-40	7300746S

### Packaging

Sterile device in blister packs of 6 pieces (assorted or individual sizes).

### Recommended procedure

#### Operating protocol:

Insert a master cone of appropriate size and taper until resistance is felt to insertion. The master taper is chosen in relation to the instrument used to shape the canal. Check radiographically for correct positioning and remove the cone. Mix the cement canal cement and insert it into the canal, then introduce the master cone into the canal again. Place the master cone, which is laterally compacted using the ELI Spreader.

During the endodontic sequence, irrigate repeatedly with hypochlorite and E.D.T.A. in order to remove all necrotic tissue and the organic component of the dental sludge.

### Storage

The blister pack should only be opened at the time of surgery and under controlled aseptic conditions.

Store at room temperature, away from moisture and direct light.

### Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.

It is recommended to always use surgical gloves for personal protection against bacterial contamination. Failure to do so may result in cross-infection.

### Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- If the package shows signs of tampering or damage due to transport or any other nature, the instruments contained therein may not be used as they may have lost sterility.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device you have supplied to the manufacturer and to the competent authority in your Member State.

### Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

### Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Sterile single barrier system

Sterilised by irradiation

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Stainless Steel

Nickel-Titanium

Manufacturer

### Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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