IT - PRO Ice

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - PRO Ice

Instant ice for cryotherapy

Description

Instant ice relieves the perception of pain and is indicated in cases of sprains, haematomas, traumas, contusions and in all cases where cryotherapy is required.

Recommended treatment for about 20 to 30 minutes.

Recommended procedure

1. Grab the bag and shake it.
2. Press towards the centre, at the point indicated, in order to break the inner bubble and thus activate the chemical reaction.
3. Shake to mix the contents.
4. Apply to the affected area.
5. Shake the bag from time to time to keep it cold for longer.

Precautions

- Always follow the medical prescription.
- Do not use if the pouch is damaged.
- Do not break the pouch or spill the contents.
- In case of accidental contact of the inner contents with the skin or eyes, rinse thoroughly with water and consult a doctor immediately.
- Do not apply the pouch to injured skin.
- Sensitisation and contact irritation may occur in patients with sensitivity and/or intolerance.
- Use only after consulting a physician in case of cardiovascular and circulatory diseases, diabetes and/or skin insensitivity (Raynaud's syndrome).
- Use the product on infants, children, the elderly and pregnant women only with a medical prescription.
- Keep away from children.
- Do not disperse in the environment. Dispose of according to current regulations.

Warnings

- For external use only.
- Disposable.
- Do not freeze.
- Contents will stain and irritate.
- The contents are harmful and should not be swallowed.
- If swallowed, consult a physician or poison control centre immediately.

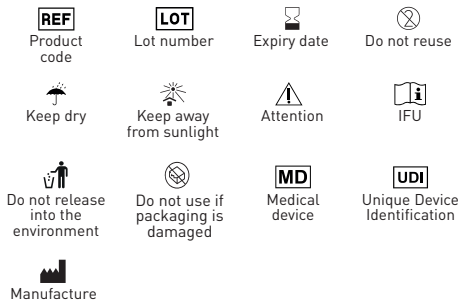
You must report any serious incidents occurring in connection with the medical device we have supplied

to the manufacturer and to the competent authority of the Member State in which you are located.

Storage

Store at room temperature, in a cool and dry place, out of children's reach, away from heat sources.

Symbols

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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