IT - Clean Flow Plus

Polvere per profilassi

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Clean Flow Plus

Powder for prophylaxis

Description

Soft and Perio powder for air-polishing devices. Designed for professional teeth cleaning using an abrasive air jet. Made from finely granulated sodium bicarbonate particles. Clean Flow Plus powder is used to remove slightly mineralized dental plaque and biofilm from supragingival teeth surface for the Soft type and subgingival teeth surface for the Perio type. The particles of the aforementioned grain sizes and the variety of flavors make cleaning highly comfortable for the patient.

Indications

Clean Flow Plus powder is designed for teeth cleaning from biofilm and dental deposits using air polishing devices.

- Removal of plaque, tartar, coffee, tea, and nicotine deposits.
- Surface preparation before placing composite restorations with filling materials.
- Surface preparation before bonding/cementing inlays, onlays, crowns, and veneers.
- Cleaning before whitening, fluoridation, and fissure sealing.
- Caries prevention.

Advantages

- **Maximum comfort and optimal performance.**
The particle size of Perio powder is on average 15 microns, used for subgingival areas, while the particle size of Soft powder is on average 40 microns, used for supragingival areas. The product is ideal for removing not only biofilm but also plaque and tartar.
- **Compatible with all brands of air abrasion systems.**
The excellent properties of the product allow it to be used with any air jet device without risk of damage. Special additives prevent particles from sticking to one another and prevent clogging and contamination of equipment.
- **Variety of flavors.**
Neutral, Mint, Blackcurrant, Cherry, Barberry, Lemon, Strawberry, Tropical.

Composition

Sodium bicarbonate, silica, sweeteners, aromas.

Precautions

- Product for dental use only.
- It should only be used by professionals who follow the procedure and instructions.
- Wear protective eyewear and gloves when handling this product.
- Keep away from non-professional users.
- Perform the treatment using circular movements without remaining in the same spot for more than 5 seconds, at a distance of 3-5 mm from the treated surface.
- Improper use may cause allergic reactions.
- Do not use on patients who have shown hypersensitivity, allergic reactions, or upper respiratory tract and infectious diseases.
- If contact with skin occurs, wash thoroughly with soap and water.
- If contact with eyes occurs, rinse immediately with water and seek medical advice immediately.
- Don not ingest.
- At the end of treatment, polish the enamel surface with

a polishing paste.

- Carefully read the instructions for use provided by the manufacturer of the air-polishing device.
- Do not use the product after the expiration date.

Warnings

- Any serious incident involving the medical device supplied by us must be reported to the manufacturer and to the competent authority of the Member State in which you are located.
- Comply with national and local regulations on disposal.

Storage

- Do not store at high temperatures or in intense light.
- Store in a cool place.
- Close the bottle immediately after use.
- Store at a temperature between 5°C and 25°C and relative humidity of 10-95%.

Packaging

Clean Flow Plus Perio - 15 micron

Bottle 300g

(code 0003960 - 0003961 - 0003962 - 0003963
0003964 - 0003965 - 0003966 - 0003967)

Clean Flow Plus Soft - 40 micron

Bottle 300g

(code 0003920 - 0003921 - 0003922 - 0003923 -
0003924 - 0003925 - 0003926 - 0003927)

Symbols

 Product code	 Lot number	 Expiry date	 IFU
 Shake well before use	 Keep dry	 Attention	 Keep away from sunlight
 Medical device	 Unique Device Identification	 Manufacturer	

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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