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Foglietti monouso

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Articulating paper

Single use sheets

Description

Articulating paper is a dental paper used to highlight occlusion contacts by creating marks on the surfaces of the corresponding teeth. Knowledge of occlusion and occlusal contacts is essential for dentists to successfully perform restorative treatments. The contact time between the paper and the tooth surface is equivalent to a few bites; the articulating ink, available in red and blue colors, will provide a two-dimensional assessment of dental occlusion that will help the doctor identify signs of occlusal instability.

Tipologie / Micron

- Articulating paper - **Red - 40 µm** (thin)
- Articulating paper - **Red - 100 µm** (medium)
- Articulating paper - **Blue - 50 µm** (thin)
- Articulating paper - **Blue - 100 µm** (medium)
- Articulating paper - **Red / Blue - 100 µm** (medium)

Method of use

- Withdraw a strip of paper from the pad, grasping it by the uncolored paper flap.
- Place the articulating paper at the dental arch manually or by a special forceps.
- Ask the patient to bite and tap the teeth on the paper until there is sufficient color transfer.

Storage

- Do not store at high temperature.
- Store in a cool, dry place.
- Do not expose to direct light and heat sources.

Precautions

- Product for dental use only.
- Should be used only by professionals who observe the procedure and directions.
- Keep away from non-professional users.
- Avoid contact of product with mucous membranes, eyes and skin. If accidental contact occurs, rinse immediately with copious amounts of water.
- Do not ingest.
- Dispose according to current regulations in your town.

Warnings

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer

and to the competent authority of the Member State in which you are located.

Packaging:

- **code 00490221**
Red - 200µm - 50 horseshoe-shaped sheets
- **code 00490225**
Blue - 200µm - 50 horseshoe-shaped sheets
- **code 00490229**
Red/Blue - 80µm - 144 horseshoe-shaped sheets
- **code 00490250S**
Blue - 50µm thin - 240 strips
- **code 00490250M**
Blue - 100µm medium - 180 strips
- **code 00490255**
Red/Blue - 100µm medium - 144 strips
- **code 00490258S**
Red - 40µm thin - 240 strips
- **code 00490258M**
Red - 100µm medium - 180 strips

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

IFU

Medical device

Unique Device Identification

Manufacturer

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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