ELASTIC DENTAL dam

Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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REV.00_10/2025

INSTRUCTIONS FOR USE

PRE-FRAMED DENTAL DAM - LATEX-FREE - POWDER-FREE

Description

Elastic Dental Dam is a medical device used by dentists to perform therapeutic procedures. It comes in the form of thin, coloured, latex-free pre-framed sheets joined to a PP film, which are placed at the beginning of the dental appointment to isolate the teeth during treatment.

Recommended procedure

- Identify the tooth concerned and make holes of approximately 2 mm in the dam sheet through which to pass the teeth to be treated.
- Use forceps to hold/widen the clamp and place it in the hole made.
- Place the dental dam with the clamp on the affected tooth. The frame should face inwards/towards the mouth.

Storage

- Store at a temperature below 40°C.
- Store in a cool, dry place.
- Keep away from heat sources and direct sunlight.

Indications for use

This product contains polysoprene, a material very similar to rubber, designed to offer its benefits even to patients who are sensitive or allergic to latex.

Precautions and warnings

- Product for dental use only.
- It must only be used by professionals who follow the procedure and instructions.
- Keep away from non-professional users.
- Single-use product. Do not reuse the device to avoid contamination or cross-infection.
- Non-sterile.
- The product cannot be used for purposes other than those for which it was designed.
- Before using the product, check the integrity of the packaging.
- Used, damaged or expired products must be disposed of in accordance with current regulations.
- Do not use the product after the expiry date.
- Any serious incident involving the medical device supplied by us must be reported to the manufacturer and to the competent authority of the country in which it is based.


Packaging (1 sheet):

- Elastic Dental Dam - Purple - Medium
- Elastic Dental Dam - Purple - Heavy

REF 00340840

REF 00340841

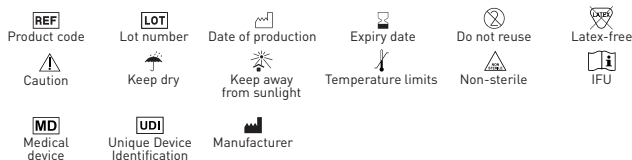Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

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