Endo Gutta Perfect

Punte di guttaperca

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Endo Gutta Perfect

Gutta percha points

Description

Gutta-percha tips are used to fill the root canal of a tooth after cleaning and shaping.

Indications for use

- Select the gutta-percha tip that best matches the last instrument used to shape the canal.
- Slowly insert the gutta-percha to the final working length.
- Insert the accessory cones into the canal next to the master gutta-percha point.
- Heat the gutta-percha and compact vertically as desired.
- Close the canal with restorative material and complete the restoration.

Precautions

- Product for dental use only.
- Non-sterile product.
- Non-reusable product.
- Do not use autoclave or heat sterilize.
- Store in a cool, dry place.
- Carefully follow the recommended storage conditions. Failure to do so will cause the gutta-percha to harden prematurely resulting in procedural delays or require re-treatment for removal.
- Do not clean or sterilize the gutta-percha, this may render the device unusable.

Warnings

- Do not apply excessive force during lateral condensation in order to avoid vertical root fracture.
- Ensure proper site preparation by removing infected tissue/bone/dentin in order to prevent infection.
- Follow proper obturation techniques to avoid procedural accidents that could lead to persistent apical periodontitis infection.
- Do not use bent or distorted gutta-percha tips.
- If a heat source is used, check the temperature to ensure that the gutta-percha melts. Incorrect temperature setting may cause procedural delays or require re-treatment.
- Do not use the product after the expiration date.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

The safety and clinical performance summary is available from EUDAMED and until that platform is fully operational, it can be requested from Dental World by sending an email to (info@dental-world.it) (Basic UDI-DI: 8050996CONIGUTTAT7)


Contraindications

Latex-sensitive patients may have an allergic reaction to gutta-percha.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Do not disperse in the environment.

Symbols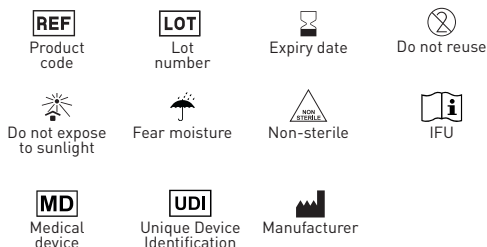

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it