# IT - Neo Alginex

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

# EN - Neo Algindex

## Dental impression alginate - Standard

### Description

A new-generation alginate used for high-precision dental impressions (where casts are required for modeling).

### Composition

- Diatomaceous earth
- Sodium alginate
- Gypsum
- Talc
- Sodium phosphate
- Food coloring

### Features and Benefits

- Class A, Type 1
- Easy to mix
- Thixotropic
- High elasticity
- High detail definition
- High tear resistance: no less than 0.38 N/mm
- Dust-free
- Avocado flavored, yellow-colored
- Elastic recovery: 95%
- Compressive deformation: 5–20%

### Recommended Procedure

1. Mix the alginate with water at room temperature (23°C) in a water/powder ratio (45 ml / 40 g) for about 45 seconds until a homogeneous mixture is obtained. **Note: The powder color will change from yellow to pink.**
2. Pour the mixture into an impression tray within 30 seconds (total time, including mixing, is 75 seconds).
3. Leave it on in the oral cavity for 1 minute and 30 seconds. After the alginate has set (and turned yellow again), remove the impression tray and rinse it under running water to remove impurities. Then pour in the plaster.

**Note: Room and water temperatures affect mixing, working, and setting times.**

Mixing time  
45 sec.

Working time  
30 sec.

Setting time  
1'30"

45 ml. 40 g

### Precautions

- For dental use only.
- This product should only be used by professionals who follow the procedure and instructions.
- Keep out of reach of non-professionals.
- Improper use may cause allergic reactions or skin and tissues irritation.
- Do not use on patients who have previously shown hypersensitivity or allergic reactions before use.
- If skin contact occurs, rinse immediately with soap and water.
- Wear protective eyewear and gloves when handling this product.
- If eye contact occurs, rinse immediately and abun-

dantly water and seek medical attention immediately.

- Do not swallow.
- Do not use the product after the expiration date.

### Storage

- Do not store at high temperatures, in direct sunlight, or near heat sources.
- Store in a well-ventilated, cool, and dry place.
- After use, reseal the product tightly in its container.
- Store preferably at a temperature ranging from -10°C to 40°C.

### Packaging

Code 001037

500g pouch

### Symbols

Product  
code

Lot  
number

Expiry date

Keep dry

Do not reuse

Keep away  
from sunlight

Attention

Non-sterile

Medical  
device

Unique Device  
Identification

IFU

Manufacturer

### Warnings

- You must report any serious incidents involving the medical device we have supplied to the manufacturer and to the competent authority of the Member State where you are located.
- Comply with national and local regulations regarding disposal.

### Warranty

Dental World will replace any product found to be defective.

Dental World assumes no liability for any damage or loss, whether direct or consequential, arising from the improper use of or inability to use the product described. Before use, it is the user's responsibility to determine the suitability of the product for their intended use. The user assumes responsibility for any improper use.

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Dental World S.r.l.