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EN

Instructions for Use

The Optimatrix Sectional system is suitable for Class II restorations and consists of preformed matrices and a clamp for fixation. Among the advantages that this system offers compared to conventional matrixes are the ease of reconstruction of contact points and the natural anatomy of the tooth, ease of use, and patient comfort.

The ends of the clamps are equipped with anatomical plastic protectors, optimizing the fit of the matrix. Optimatrix Sectional clamps are available in two models: **Large**, suitable for cases requiring the production of contact points with greater pressure and tooth separation, and **Small**, suitable for clinical cases requiring less pressure and tooth separation.

Method of use

Isolate the operating field with a rubber dam or other isolation technique. Choose from 3 matrix sizes, selecting the one that best fits the cavity to be restored. Install the matrix in the proximal area to be restored.

In the case of a MOD cavity, it is recommended to restore one face at a time. Install the wedge. Elastic wedges improve the fit of the matrix. In this case, plastic protectors are not used and the clamp is positioned in the perforations of the elastic wedge. Insert the appropriate clamp depending on the clinical case. Palmer-type hook forceps can be used to open and insert the clamp. The use or non-use of the clamp and plastic protectors and the method of application depend on personal preference and the clinical situation. Once the restoration is complete, first remove the fixing clamp, then the wedge, and finally the matrix. To obtain consistent contact points, it is recommended to separate adjacent points from each other using wedges or a fixing clamp, from the time of anesthesia until the installation of the matrix.

Warnings, precautions, and restrictions

- Professional use.
- Matrixes are for single use only. Reuse may cause cross-contamination.
- If the patient has an established allergy to any of the components, the product should not be used.
- When inserting the clamp, keep your finger pressed on it until the clamp is removed, ensuring adjustment and avoiding slippage. Care should be taken when inserting the clamp because of its flexibility.
- Make sure the product is fully inserted into the clamp.
- Avoid leaving the clamp open in the clamps for a long period of time.
- It is recommended to protect the papilla by any method to avoid accidents.
- It is recommended that the use of the product should always be accompanied by absolute isolation of the operating field, as loose objects could be accidentally ingested or aspirated.
- The clamp must be used with plastic protectors for proper fitting of the die. The die and protectors must be sterilized separately.
- You must report any serious incidents that have occurred in connection with the medical device we supply to the manufacturer and to the competent authority of the member state in which you are located.

CLAMP / PLASTIC PROTECTORS Cleaning

- Carry out cleaning with the help of water and detergents with 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pure pH between 6.5 - 7.5.
- Prepare the solution according to the manufacturer's directions.

- For greater effectiveness, it is recommended that the temperature be between 40°C and 55°C.
- Immerse the products and leave them in contact with the solution for 5 minutes. Remove the parts and rinse with running water for at least 10 seconds.
- Dry the parts with paper towels or disposable napkins.

Sterilization

The clamp and protectors must be sterilized separately in an autoclave at 134°C according to the timing specified for this cycle. Sterilization may cause a slight change in the color of the protectors and reinforcements, but this does not affect their functionality.

Inspection

Inspect the product before each use. Note if there is any damage (breaks/cracks/rust) that may prevent it from functioning properly, so do not use it in such cases.

Limitations of reprocessing

This is a reusable device. Repeated reprocessing, carries the risk of damage from use.

Composition

- Matrixes: stainless steel.
- Clamp: titanium.
- Clamp reinforcement: thermoplastic.
- Protectors: plastic.

Store in a dry, cool place, away from sunlight or direct light, and dispose of in accordance with local regulations.

Symbols

Product code

Lot number

Date of manufacturing

Keep dry

Keep away from sunlight

Not-sterile

Attention

Autoclavable

IFU

Medical device

Unique Device Identification

Manufacture

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

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Warnings, precautions and restrictions

- Professional use.
- Matrices are for single use only. Reuse may cause cross-contamination.
- If the patient has an established allergy to any of the components, the product should not be used.
- When inserting the clamp, keep your finger pressed on it until the clamp is removed, ensuring adjustment and avoiding slippage. Care should be taken when inserting the clamp because of its flexibility.
- Make sure the product is fully inserted into the clamp.
- Avoid leaving the clamp open in the clamps for a long period of time.
- It is recommended to protect the papilla by any method to avoid accidents.
- It is recommended that the use of the product should always be accompanied by absolute isolation of the operating field, as loose objects could be accidentally ingested or aspirated.
- La pince doit être utilisée avec les protections en plastique pour un ajustement correct de la matrice. La matrice et les protections doivent être stérilisées séparément.
- You must report any serious incidents occurring in connection with the medical device we supply to the manufacturer and to the competent authority of the member state in which you are located.

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- Perform cleaning with the help of water and detergents with 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pure pH between 6.5 - 7.5.
- Prepare the solution according to the manufacturer's directions.
- For greater effectiveness, it is recommended that the temperature be between 40°C and 55°C.
- Immerse the products and leave them in contact with the solution for 5 minutes. Remove the parts and rinse with running water for at least 10 seconds.
- Dry the parts with paper towels or disposable napkins.

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Dental World S.r.l.