IT - Perfect 4 Dam

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Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Perfect 4 Dam

Rubber dam clamp

Description

Perfect 4 Dam is the new line of rubber dam clamps, used to be positioned over a tooth in order to subsequently secure the rubber dam and retract the tissue of the area to be treated.

Recommended Procedure

1. Remove the clamp from the packaging, then wash and sterilize it before use.
2. Before use, inspect the clamp to verify that it has the correct tension and is not deformed; if this is the case, discard any clamps showing anomalies.
3. Apply a safety ligature (dental floss) around the clamp to prevent it from being ingested in the event that it becomes detached or breaks [Fig. A].
4. Try the clamp to ensure that there are four points of contact with the proximal surfaces of the tooth. Using the winged technique, position the clamp in the hole made in the dam so that its wings keep the hole open.
5. With the aid of rubber dam clamp forceps, the dam and clamp can be positioned over the tooth. Then slide the rubber dam over the tooth, passing it outside the wings of the clamp.

Figure A

Cleaning and Care

- It is recommended to thoroughly wash and rinse the clamps following the procedure.
- Failure to clean them may reduce the life of the clamp and cause stains or corrosion.
- Before ultrasonic cleaning, rinse and remove any material residues.
- Use a neutral pH cleaning solution and follow the instructions provided by the manufacturer of the ultrasonic cleaner.
- Rinse the clamps to remove detergents.
- Check that all material residues have been removed from the clamps; otherwise, repeat the washing procedure.
- Allow the clamps to dry.
- To ensure maximum durability, do not bend or deform the clamp. Avoid allowing instruments used during the procedure to damage or scratch the product.
- If techniques involving the use of sodium hypochlorite are employed, make sure that the clamps are rinsed with water immediately after use. Prolonged contact with sodium hypochlorite may cause corrosion, significantly shortening the service life of the clamp.

Storage

Store in a safe place out of the reach of children.

Precautions and Warnings

- Product for dental use only.
- It must be used only by dental personnel who follow the procedure and instructions.
- Keep away from non-professional users.
- Deformed, worn or cracked clamps must be replaced.
- Do not ingest.
- The service life of the clamps depends on the number of uses and the care taken during application, cleaning and storage.
- If detergents or solutions with neutral pH are not used, the clamps may be damaged.
- Any serious incident that has occurred in relation to the me-

dical device supplied by us must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

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Symbols

 Product code	 Lot number	 Date of manufacturing	 Attention
 Not-sterile	 Suitable for autoclave sterilisation	 IFU	 Medical device
 Unique Device Identification	 Manufacturer		

Warranty

Dental World will replace any product found to be defective. Dental World accepts no liability for any damage or loss, whether direct or consequential, arising from the careless use of or inability to use the product described. Before use, it is the user's responsibility to determine the suitability of the product for their intended use. The user assumes responsibility for any careless use.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

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