Flexo File

Strumenti canalari manuali

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Flexo File

Manual Canal Instruments

Description

Flexo Files are super flexible stainless steel files suitable for the preparation of irregular canals.

Indications

To be used only in dental clinics or hospitals by qualified personnel.

Application field

For catheterization and shape definition.

Measures

FLEXO FILE		
	21 mm	25 mm
○ 15	144517182	144517242
● 20	144517192	144517252
● 25	144517202	144517262
● 30	144517212	144517272
● 35	144517222	144517282
● 40	144517232	144517292
15-40	144550712	144550732

Packaging

Blister 6 pcs (assorted or single sizes).

Recommended procedure

Operative protocol:

Use the Flexo File for opening and shaping the root canal. In stainless steel, they guarantee greater flexibility for cleaning canals. The triangular section facilitates the removal of residues. Work in quarters of rotation, then remove the tool.

Storage


Store the product at room temperature away from direct light.

Cautions

- Instruments are disposable.
- Repeated use may not be effective and/or cause damage/ inconvenience to the patient.

- Do not use the blister if opened or damaged.

Symbols

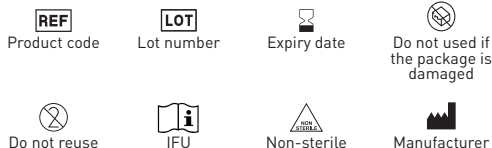Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

Dental World S.r.l.

Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it