IT - Future Pilot N.H.A. - Future Pilot H.A.

Strumenti canalari meccanici

Descrizione

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EN - Future Pilot N.H.A. - Future Pilot H.A.

Mechanical root canal instruments

Description

Future Pilot H.A. and Future Pilot N.H.A. are Ni-Ti instruments available in Gold, Blue and Coated, with thermal activation. They offer excellent flexibility by easily adapting to the anatomy of most root canals. Instruments with progressively variable taper allow you to work with greater confidence in establishing the glide path.

Indications

To be used only in a dental practice or hospital environment by qualified personnel.

Field of application

Glide path creation.

Measures

Future Pilot NHA						
	Ø	%	N/cm	RPM	21 mm	31 mm
○ 016	15	2-6	2,5	300	166900551	166900561
					166900571	

Future Pilot Heat Activation						
	Ø	%	N/cm	RPM	21 mm	31 mm
○ 016	15	2-6	2,5	300	166900751	166900761
					166900771	

Future Pilot Heat Activation Gold						
	Ø	%	N/cm	RPM	21 mm	31 mm
○ 016	15	2-6	2,5	300	166900660	166900670
					166900680	

Packaging

Non-sterile device in blister pack of 6 pieces.

Recommended procedure

Operating protocol:

- Locate the canal.
- Use a coronal portion opening tool (Future Flayrel).
- Probe with an ISO 010 or 015 manual file.
- Work gently with the Future Pilot H.A. or Future Pilot N.H.A. rotary instrument (ISO 016) in order to obtain a reproducible glide path before canal widening and to reduce the workload of the shaping instruments and risk of breakage.
- Flush the canal using sodium hypochlorite and viscous chelator as lubricant.
- Then follow the operative sequence most suitable for the case to be treated.

Throughout the endodontic sequence, repeatedly irrigate with hypochlorite and E.D.T.A. to remove all necrotic tissue and the organic component of the dentinal sludge.

Storage

Store at room temperature, away from moisture and direct light.

Precautions

Before use, it is recommended to check with patients for any allergy to raw materials.
It is recommended to always use surgical gloves for

personal protection against bacterial contamination. Failure to do so may result in cross-infection.

Warnings

- Single-use device, re-use is not recommended to limit problems with cross-infection and/or breakage.
- Do not use the blister if opened or damaged.
- Never force the instrument into the canal.
- Do not use after the indicated expiry date.
- You must report any serious incidents occurring in relation to the medical device we have supplied to the manufacturer and to the competent authority in your Member State.

Contraindications

Do not use in patients with known allergic sensitivity to raw materials.

Symbols

Product code

Lot number

Expiry date

Do not reuse

Keep dry

Keep away from sunlight

Non-sterile

IFU

Packing unit

Do not use if packaging is damaged

Medical device

Unique Device Identification

Nickel-Titanium

Manufacture

Warranty

Dental World shall replace the product if it proves to be defective.

Dental World does not accept liability for any damage or loss, direct or consequential, resulting from careless use or inability to use the product described. Before use, it is the user's responsibility to determine the appropriateness of the product for its use. The user assumes responsibility for use careless use.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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PREMISE

For hygiene and health safety purposes, all instruments must be cleaned, disinfected and sterilized before each use to prevent contamination. This applies to both first and subsequent uses. Instruments must undergo this procedure for all uses, unless they are labeled "single-use."

APPLICATION AREA

Disinfection and sterilization must be performed before each use and/or reprocessing procedures on the following instruments:

cutting instruments, manual or mechanical such as endodontic instruments, files, probes, reamers, plugger inserts, spreaders, compactors etc.

DISINFECTION, CLEANING AND STERILIZATION

Disinfection and sterilization procedure for root canal instruments and implantable root canal devices.

STERILIZATION PROTOCOL FOR ROOT CANAL INSTRUMENTS

The sterilization protocol for root canal instruments manufactured by DENTAL WORLD srl involves a multi-step sterilization process:

- preliminary disinfection: immerse the instruments in a 2.5% to 5% sodium hypochlorite solution for 5 minutes;
- wash and dry the instruments carefully;
- package the instruments in autoclave bags;
- sterilize the bags containing the instruments in a steam autoclave at a temperature of 134°C (sterilization time varies depending on the autoclave used).