Frese endodontiche

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

Endodontic drills

Description

The endodontic drill is an instrument used to prepare the root canal for the insertion of the corresponding post.

Recommended procedure

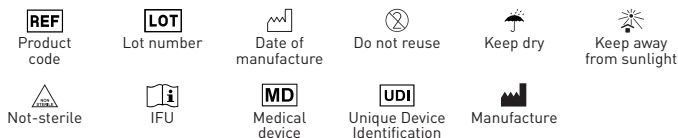
1. Before being used, the drills must be disinfected, cleaned and sterilised.
2. Place the drills in the appropriate disinfection trays; use non-corrosive disinfectants.
3. Clean the cutters in ultrasonic baths.
4. Dry.
5. Before sterilisation, place the drills on special holders.
6. Sterilise with moist heat at 134°C for 5 minutes in cycles complying with the requirements of ISO 17665-1.
7. After sterilisation, place the bur in the appropriate containers in a clean, dry place.
8. Use the burs on a contra-angle handpiece.
9. Use water jets during all phases of use, for good cooling.
10. Avoid levering the cutter during use.
11. Observe the maximum operating speed (15.000 rpm).
12. Cutters must be kept in perfect working order; damaged or particularly worn cutters must be replaced; it is recommended not to exceed 30 reuses.
13. For temporary storage of spent MD, use the appropriate containers for hospital waste; for disposal follow the Waste Management Regulations of the local managing body (Municipality, Municipal authority, etc.), in relation to special hospital waste.

Warnings

- Non-sterile device.
- Product for dental use only.
- The drills should only be used by professionals observing the procedure and indications, such as dentists and specialists in odontostomatology.
- The maximum number of cycles is 30. Using more than this may result in harm to the patient due to loss of drill performance.

You must report any serious incidents occurring in connection with the medical device we supply to the manufacturer and to the competent authority of the Member State in which you are located.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta [BA]
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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