IT - Eclipse

Bottoni linguali

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Eclipse

Lingual buttons

Description

The lingual button is a device with a meshed base designed with a head suitable for applying any type of elastic or ligature wire that can be directed differently. The neck is not very high for patient comfort, and the base has beveled corners so as not to create irritation.

Indications for use

Passive elements for attaching aids such as elastic bands, chains, springs.

Recommended procedure

- Apply the buttons to be cemented directly onto the tooth.
- Apply the buttons to be soldered to the bands.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, car-rots, etc.
- In case of allergic reaction, remove the device immediately.
- The product contains nickel so in case of suspected or established allergy/sensitivity of patient do not use in any way.

Comply with national disposal regulations.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

Contraindications

- Poor oral hygiene.
- Known allergies to any of the product's components.
- General/underlying conditions that preclude orthodontic treatment.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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