IT - Eclipse

Molle ortodontiche

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Eclipse

Orthodontic springs

Description

Orthodontic springs are used to open and maintain dental spaces. Springs that contain steel eyelets allow easy positioning of the spring. Depending on the type of spring, a medium to light force can be applied.

Indications for use

Orthodontic springs are used to move teeth by tensile and pressure forces.

Recommended procedure

- Cut the spring to the desired length and apply it to the bow. Compress the spring until the coils are closed. Apply it in the gap of the bracket or between a stop on the arch and the bracket.
- For springs with eyelets, hook it onto the hook of the brackets and buccal tubes or the stops applied on the arch. Do not overstretch the spring beyond the allowed length so so that it does not lose its antagonistic force.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, carrots, etc.
- Tell the patient to eat and clean teeth carefully so as not to risk the springs coming off.
- Do not leave the springs activated indefinitely or for an excessively long period to prevent them from breaking off inside the mouth.
- In case of allergic reaction, remove the device immediately.
- The product contains nickel so in case of suspected or established allergy/sensitivity of patient do not use in any way.

- Comply with national disposal regulations. You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

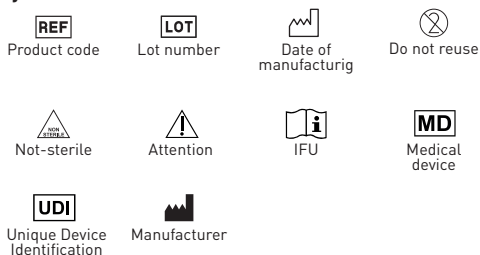
Contraindications

- Poor oral hygiene.
- Known allergies to any of the product's components.
- General/underlying conditions that preclude orthodontic treatment.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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