IT - Eclipse

Archi ortodontici in Acciaio (SS)

Nichel-titanio (Ni-Ti)

Descrizione

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EN - Eclipse

Orthodontic archwires in stainless steel (SS) and nickel-titanium (Ni-Ti)

Description

Orthodontic arches are a device intended for use in orthodontic treatment with the task of exerting forces to move the teeth and bring them into the correct position.

Indications for use

Orthodontic arches are used as force elements for tooth movement.

Recommended procedure

- Introduce the arch into the slot of the brackets and buccal tube and secure by elastic or wire ligation.
- If self-ligating brackets are used, close them according to the manufacturer's instructions.
- Use tools or pliers without sharp edges and grooved inserts to grasp and hold the bows.
- Repeated excessive bending or the use of sharp bending pliers can cause the bow to break.

Warnings

- The product should be used by qualified pro-fessional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard sub-stances such as hard candies, ice, car-rots, etc.
- In case of allergic reaction, remove the device immediately.
- The product contains nickel so in case of su-s-pected or established allergy/sensitivity of the patient do not use in any way.
- Comply with national disposal regulations.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

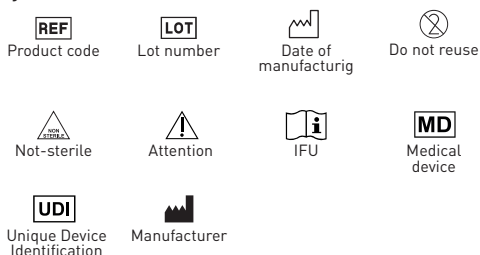
Contraindications

- Insufficient oral hygiene on the part of the pa-tient.
- Known allergies to any of the product's com-ponents.
- General/underlying diseases that prevent or-thodontic treatment.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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