IT - Eclipse

Tubi buccali ortodontici

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Eclipse

Orthodontic buccal tubes

Description

Buccal tubes are devices used in orthodontics to align and promote proper tooth positioning and are often used in conjunction with other orthodontic devices prescribed by a professional orthodontist.

Indications for use

Made of stainless steel, the tubes are placed on the patient's teeth with an adhesive and then connected to each other through arches and bands for the purpose of facilitating precise tooth movements.

Recommended procedure

1. Clean the tooth surface to remove excess tar, stains, etc., then rinse with water and dry. Use mouth openers, cotton rollers and suction saliva to facilitate etching.
2. Apply etchant to the surface of the tooth on which the tube is to be placed.
3. Rinse and dry to remove excess acid and debris. At this point, the tooth surface will be opaque.
4. Draw the reference line of tube attachment placement on the tooth surface using a positioner.
5. Apply primer to the etched tooth surface, creating a thin layer. Then apply an appropriate amount of adhesive to the base plate, place the tube on the tooth surface and press it lightly. Remove adhesive residue in time and keep the tooth surface smooth around the tube.
6. Polymerize the adhesive with a curing light so that the tube is firmly attached.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, carrots, etc.
- In case of an allergic reaction, remove the orthodontic retainer immediately.
- Orthodontic buccal tubes are contraindicated

for those who are allergic to chromium and nickel.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

Contraindications

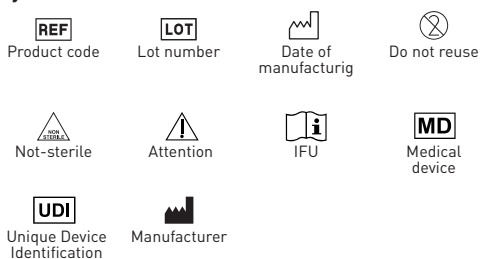
Heavy smoke, tobacco and alcohol abuse, systemic blood diseases, uncontrolled diabetes.

The device should not be used for patients with periodontitis.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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