IT - Eclipse

Elastici intraorali in lattice

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Eclipse

Intraoral latex elastics

Description

Latex elastics are used in the field of orthodontics to correct various types of malocclusions.

Indications for use

Intraoral elastics are indicated for orthodontic treatment in patients of all ages under prescription from an orthodontist.

Recommended procedure

1. Select the correct size of latex rubber band according to the orthodontic method used.
2. Use orthodontic forceps to apply the rubber band to the affected tooth.
3. The duration of use is at the discretion of the orthodontist.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, carrots, etc.
- In case of an allergic reaction, remove the orthodontic retainer immediately.
- During oral hygiene, use a soft toothbrush and brush gently.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

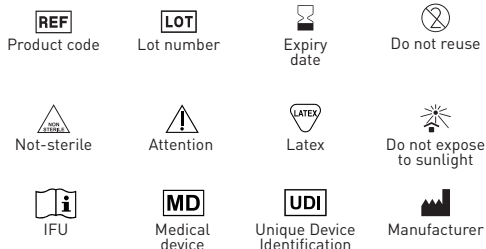
Contraindications

- These products should not be used in patients allergic to latex.
- Discontinue use of the product if an allergic reaction occurs.
- Reuse is not permitted, in which case cross-contamination may occur.
- Do not use after the expiration date.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Protect from sunlight or direct light.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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