IT - Eclipse - Elastomeri

Catenelle - Separatori - Legature

Descrizione

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Via Antichi Pastifici, 15 - 70056 Molfetta (BA)

Tel. +39 080 3381004 - Fax. +39 080 3386617

e-mail: info@dental-world.it / web: www.dental-world.it

EN - Eclipse - Elastomers

Chains - Separators - Legatures

Description

Orthodontic elastomers, made of high-density polyurethane, are used to apply forces to teeth and are used in orthodontics to correct various types of malocclusions.

Indications for use

- Elastic separators are used to open interproximal spaces between teeth and are ideal for creating spaces between molars for placement of orthodontic bands.
- Elastic ligatures are used to secure the arch to the bracket and are usually changed at each adjustment.
- Elastic chains are capable of exerting light, continuous forces for optimal tooth movement.

Recommended procedure

- Select the elastomer size (in the case of: chains, ligatures and separators).
- Use a suitable orthodontic forceps to position the device in use.

Warnings

- The product should be used by qualified professional orthodontists.
- For dental use only.
- Do not reuse the product.
- Non sterile product.
- Instruct patients not to chew or bite hard substances such as hard candies, ice, carrots, etc.
- During oral hygiene, use a soft toothbrush and brush gently.

You must report any serious incidents occurring in relation to the medical device you supply to the manufacturer and to the competent authority of the Member State in which you are located.

Contraindications

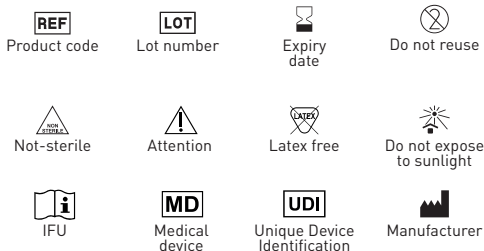
- Discontinue use of the product if an allergic reaction occurs.

- Reuse is not permitted, in which case cross-contamination may occur.
- Do not use after the expiration date.

Storage

- Store at room temperature.
- Store properly in their own packaging.
- Protect from sunlight or direct light.
- Do not disperse in the environment.

Symbols

Warranty

Dental World will replace product that is proven to be defective. Dental World does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risks and liability in connection therewith.

 Dental World S.r.l.
Via Antichi Pastifici, 15 - 70056 Molfetta (BA)
Tel. +39 080 3381004 - Fax. +39 080 3386617
e-mail: info@dental-world.it / web: www.dental-world.it

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